COLD & WARM

POST-BIRTH RELIEF PAD

Microwave Instructions

	25s / 700W	19s / 1,000W	16s / 1,250W
---	-------------------	---------------------	---------------------

EN

Use cold for relief from pain and discomfort after childbirth, c-section or from haemorrhoids. Use warm at the abdomen for soothing relief from postnatal uterine cramping.

For Cold Therapy:

Freeze pack for 4 hours before use.

For Heat Therapy:

Microwave for time indicated at the first page. Gently massage pad to evenly distribute heat as indicated by colour-changing pearls and test pack prior to use. Add time in 5 second increments until desired temperature is reached.

For Heat Therapy -

Water Bath Instructions:

Fill a large pot with enough water to fully submerge the pack without touching the bottom of the pot. Bring water to a boil and remove pot from heating surface. **Do not immerse pack while water is being heated.** Use a blunt, nonmetallic tool to submerge pack into the water for 1-2 minutes. **Do not leave pack unattended while heating.**

If pack has not reached desired temperature, submerge again (in same water) for an additional minute. Use blunt, non-metallic tool to remove from water.

Do not allow the pack to come in contact with the pot or heat source for prolonged time, excessive contact may rupture the pack. Towel dry pack before applying. Always inspect the pack for signs of damage or leaking prior to use and discard if damage is found. Always test the pack temperature before application and allow to cool if needed.

Colour is not indicative of pack temperature. Apply pack for 20 minutes or as recommended by your doctor.

Sleeve Instructions

After cooling or heating, place the pack into the disposable sleeve. Sleeves may be used with cold or warm pack. Refill pack of disposable sleeves available separately.

Cleaning Instructions:

After each use, thoroughly clean pack with mild soap and water. Rinse thoroughly until all soap and residue is removed. Allow to air dry completely prior to use. We do not recommend the use of ethanol, phenol, or chlorine-based products

Suggested Uses

Perineal pain and haemorrhoids Cold therapy, applied to the affected area can help reduce swelling and relieve pain. Apply the pack for up 20 minutes at a time allowing at least an hour in between uses.

C-Section relief

Cold therapy applied to the affected area can help reduce swelling and ease discomfort. Use for up to 20 minutes at a time or as directed by your physician.

Postnatal uterine cramping

Heat therapy, applied for twenty minutes at a time with at least an hour in between or as directed by your physician may help relieve the discomfort of the cramping, as your uterus begins to return to its original size.

CAUTION: Always test pack temperature prior to application. Prolonged application may cause injury, including burns. Do not use on infants. Do not use if you have poor circulation. Do not use heat therapy on stitches or an open wound. Do not fall asleep using pack. Keep out of reach of children. For extreme, prolonged, or chronic pain, contact a physician. Use only as directed. Heating beyond suggested time may damage or destroy pack. Do not puncture pack and/or ingest contents. For external use only. Do not place disposable sleeves in microwave.

		EMERGO EUROPE Prinsessegracht 20, 2514 AP The Hague, The Netherlands
		MD Medical Device - Serious events experienced from using this product should be reported immediately to The Hygenic Corporation and to the local competent Authority.
		Distributed by: Lansinoh Laboratories UK Ltd, Ground Floor, West Mayesbrook House, Lawnswood Business Park, Leeds, LS16 6QY, UK
		Lansinoh® is a registered trademark of Lansinoh Laboratories Inc. TheraPearl® and ColorSmart™ are registered trademarks of Therapearl LLC © 2021 Lansinoh Laboratories Inc. This product and/or packaging may be protected by pending or issued U.S. or foreign intellectual property, including (but not limited to) patents, Industrial Design Restrictions, and/or Registered Community Designs. See www.therapearl.com/ patent for specific details.
		 The Hygenic Corporation 1245 Home Avenue, Akron, OH 44310, USA 1-877-732-7509
		 Consult instructions for use
		 Made in China PE69130-69140IFO321 15138 P11446-R01

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